

Test Implant-Guide®

Implant-Assistant®

Considerable reduction in surgery time

Precise installation of implants in the pre-planned position

Safe surgery

More stable post-operative period

High esthetic and functional result



To find out more about this high precision dental implantation surgery technology, the **Implant-Assistant team** is offering you the opportunity to make an individual mucous-supported, bone-supported, or teeth-supported **Implant-Guide®** guide.

You can make a single order for a test guide without purchasing Implant-Assistant® software.

Before starting CT analysis we highly recommend reading the following:

- **Implant-Guide® variants** (Variants of Implantation Guides)
- **X-ray templates** (Making Radiopaque Guides)
- **CT-scan** (Computed Scanning Procedures)

STEP 1 — UPLOADING ANALYSIS

Save the CT analysis results in the DICOM format and archive them. Then contact the Implant-Assistant team to upload them to the server at www.implant-assistant.com. You can upload the data by yourself using any of the free file servers, for example:

<http://www.rapidshare.com>

<http://www.jetbytes.com>

Email your order to mail@implant-guide.ru

The order should contain the following info:

- Positions for implant installation;
- Implant brands you are planning to install;
- Implant-Guide variant that you want to use;
- Link to archive with results of the CT analysis.

STEP 2 — PLANNING SURGERY

The CT scan data will be analyzed, our doctor will arrange the implants and we will send you the link to the Implant-Assistant project containing the arranged implants by email. You have to download and install the free **Implant-Assistant Viewer** to view this project.

You will open and view the project. If you agree with the surgery plan, then you only pay for the cost of the guide, the required components and delivery.

The Dental Implantation Centre team will analyze the CT scan for the Implant-Assistant Planner program free of charge!

STEP 3 — PERFORMING SURGERY

The ready-made guide with the required components will be delivered to you by courier or express-delivery.

Recommendations on how to work with the guide can be read in the “**Implant-Guide® variants**”, and “**Implant-Guide® components**” memos.

Perform the operation using **Implant-Guide®** and see the results.



Implant-Assistant®

*Implant-Assistant team
wishes you successful
operations!*